

## **US** Troubleshooting Guide Diago 504 US English

### Products

Wall Cabinet Lift - DIAGO

The Diagonal lift wall cabinet lift unit is designed and tested to achieve optimum operation reliability and long life, provided that the operation, maintenance and inspections are carried out in accordance with the prescribed instructions. If in spite of this a fault appears you can get guidance about what to do according to the fault finding list below.

Remove the load from the cabinet before fault finding and repair work is carried out. Fault finding, service and repair work shall be made by competent persons only.

Symtom	Action
<b>The cabinet lift does not move when a control signal is given</b>	<p>Check that the fuse for the mains feed is not damaged.</p> <p>Check that the mains plug is connected.</p> <p>Check that the safety trip panel is not actuated. Check that the wires are properly fastened in the control box.</p> <p>If the cabinet lift still does not move contact an authorized technician and/or your supplier.</p> <p>This shall as first action, after above check points, check that all cables are connected and undamaged, and that the green light on the control unit is lighting. If it is not lighting the control unit is probably defect. Order a new unit from Granberg.</p>